







AlcoBlow®

The AlcoBlow® is a rapid screener which detects the presence or absence of alcohol in the body. This easy-to-use, portable device is invaluable to the Police at the roadside, to an employer in the workplace or in any environment where multi-screening is a necessity.

- O Easy-to-use: just 2 control buttons
- Reliable analysis in seconds
- Simple display system: result shown on a single LED
- No mouthpiece required: no physical contact
- Dual mode: active and passive



UP TO SIX TEST PER MINUTES CAN BE ACHIEVED!

A Worldwide Company ... For a Safer World ...





+6585692024

Simple Operating Procedure





Accessories:

Protective silicone skin

(yellow or black)





Specifications:

Analytical principle: Lion fuel cell sensor

Response Time:

Within 3 seconds of sampling

Recovery Time:

2 seconds at zero alcohol 20 seconds at 0.50mg/l BrAC

Operating Temperature Range:

-5°C to +40°C

Power Supply:

4 x AA batteries (5,000 breath tests)

Weight

Approx. 300g (including batteries)

Dimensions:

265 x 45 x 40mm



Lion Alco Blow is simplicity itself to use! Switch on, Wait a few moments for the STATUS light to show green, then start sampling breath!









Consumables:





Up to six tests per minute can be achieved!

Very economical operation, *no disposable mouthpieces are required.* The subject simply blows into a cone at the end of the instrument.

The test result is indicated by three coloured lights within seconds.

USER CONTROLS

www.bshop.com.sg

www.bshop.com.sg

The Lion AlcoBlow® has two push buttons and two LED indicator lights. These are clearly labelled, easy to use and protected for use under severe operational conditions.



AlcoBlow

On/Off

On/Off switch - press and release once to switch on, then once again to switch off.

If the unit is left unused for 2 minutes it switches itself off automatically, to save battery life.

PASSIVE switch - This is used to obtain a breath sample when it has been observed that the subject has difficulty in blowing or is unco-operative. Simply press and release to take a sample from in front of the subject as the subject is breathing or talking normally.

The PASSIVE switch can also be used to determine whether an unconcious person's breath contains alcohol by holding the AlcoBlow close to and in front of the persons mouth or nose.

STATUS light: has two colours:

red wait

green ready for breath sample. Also flashes to indicate "Low Battery"

ALCOHOL light: three colours show the approximate alcohol level in the breath sample:

green zero(.000 - .010% BAC)

| amber | low (.010 - .020% BAC) | red | high (over .020% BAC)

www.bshop.com.sg